

Short instructions for disinfection of measuring prisms and contact lenses


English




NOTE!
Excerpt from Haag-Streit instruction for use "Cleaning and disinfection". Complete instructions for use: www.haag-streit.com



FORBIDDEN!
Never use damaged measuring prisms or contact lenses.

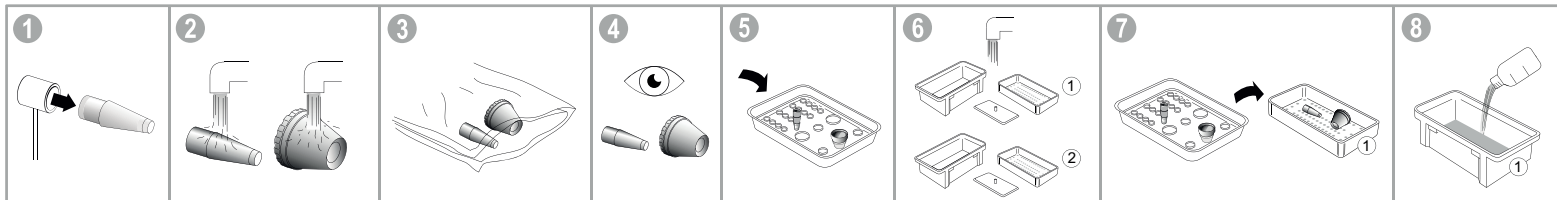

- Remove the measuring prisms from the holder.
- Clean the measuring prisms and/or contact lenses under cold drinking water until no more soiling is visible.
- Dry measuring prisms and/or contact lenses with a new, lint-free disposable cloth.
- Perform visual checks for cleanliness and perfect working condition.
- Place the cleaned measuring prisms and/or contact lenses in a clean plastic container.
- Clean and disinfect the baths, the bath inserts and the weights for immersing.
- Place all measuring prisms and/or contact lenses to be disinfected in the bath insert.
- Take the first bath and add at least as much disinfecting solution as necessary to fully immerse the measuring prisms and/or contact lenses.

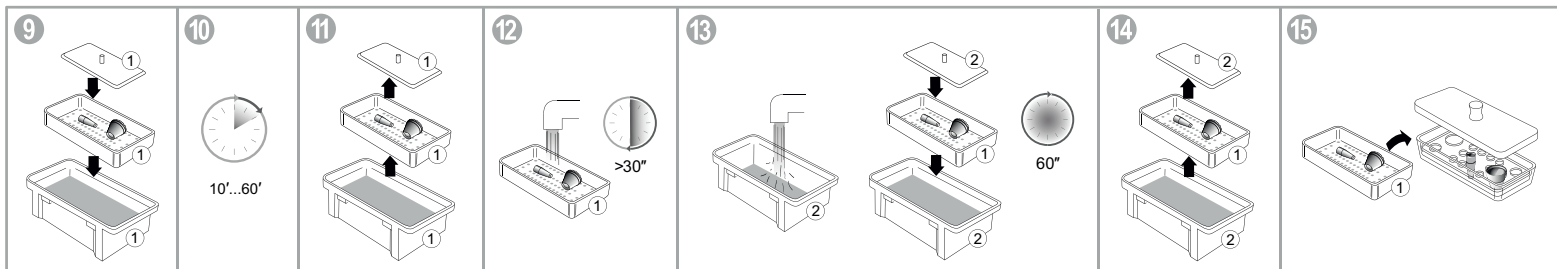

- Immerse the bath insert with the measuring prisms and/or contact lenses in the disinfectant solution and place the weight for immersing so that all measuring prisms and/or contact lenses are fully immersed.
→ **All measuring prisms and/or contact lenses must be completely immersed in the disinfectant solution.**
→ **As the measuring prisms tend to float in the solution, they must always be weighted with the weight for disinfection bath.**
- Allow the disinfectant sufficient time to work as specified in the disinfectant manufacturer's instructions.
- Once the exposure time has expired, remove the bath insert with the measuring prisms and/or contact lenses.
- Rinse the measuring prisms and/or contact lenses in the bath insert under running, cold drinking water thoroughly for at least 30 seconds.
- Immerse the bath insert with the measuring prisms and/or contact lenses in a second bath filled with cold drinking or distilled water. Place the second weight for immersing so that all measuring prisms and/or contact lenses are fully immersed. Set for 60 seconds.
- Remove the bath insert with the measuring prisms and/or contact lenses.
- Store the cleaned and disinfected measuring prisms and/or contact lenses in a clean and disinfected container until their next use.



NOTE!
The disinfectant must be replaced as follows:
• When visibly soiled
• Once a day or as specified by the manufacturer
List of disinfectants: www.haag-streit.com



FORBIDDEN!

- Disinfection with alcohol
- Cleaning with acetone
- Disinfection using UV radiation
- Sterilization using steam or ethylene oxide
- Temperatures above 60°C

HS HAAG-STREIT
DIAGNOSTICS



HAAG-STREIT AG
Gartenstadtstrasse 10
3098 Koenig, Switzerland
www.haag-streit.com