

QUICK REFERENCE GUIDE MR Q SUPINE (1)


SAFETY REQUIREMENTS



An appropriate safety glasses is required for all present persons when the System is in READY mode.


The MR Q has a built-in safety filter to protect the user

PRODUCT DESCRIPTION

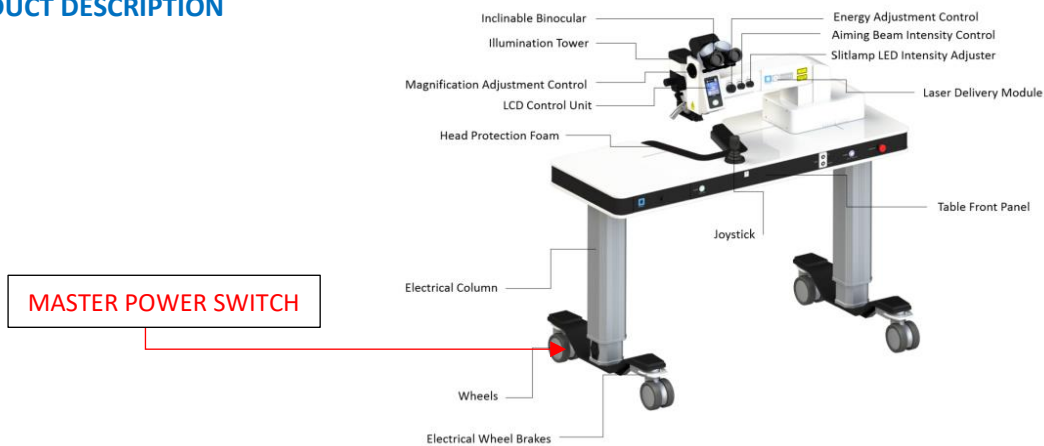

MASTER POWER SWITCH

MAIN TABLE CONTROLS

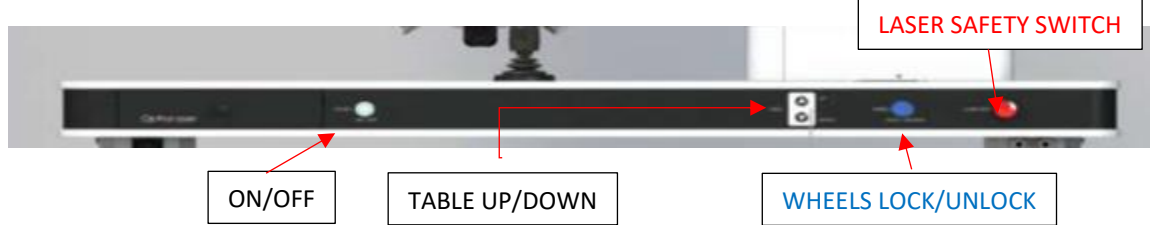

ACCESSING THE MR Q

STEP 1	STEP 2	STEP 3
System activation	Enter PIN code	Standby mode


The laser does not have a KEY SWITCH, instead has a KEYPAD, a four number password is setup by the user. **The Default Code is in the LASER KEY CABINET.**

To unlock the laser the user must type the code followed by ----- >

BEFORE USING THE LASER

User must calibrate the binoculars, to assure the laser focus and visualisation are on the same focal plane.

Position the laser head in the "Zeros" using the joystick

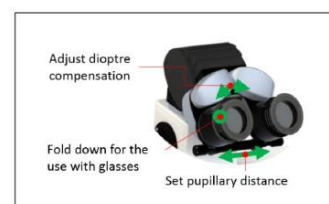

Switzerland

Meridian AG
Bierigulstrasse 7
3608 Thun

Finland

Meridian Medical Oy
Eläntie 5
01510 Vantaa

Slovenia

Meridian Medical d.o.o
Plomljeva 8
1210 Ljubljana

Distributor Information


OPHTHALMALOGIC / MERIDIAN
Rick Behrens
0431 002 302

QUICK REFERENCE GUIDE MR Q SUPINE (2)

CONTROL PANEL

By pressing the icons, the setting is modified or reset.

FOR CAPSULOTOMIES & IRIDOTOMIES YAG TREATMENTS USE OFF-SET P100 or P200

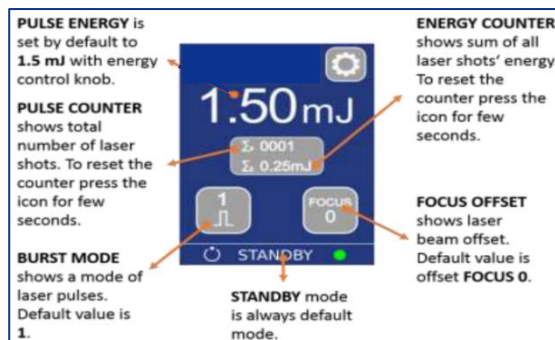

LASER CONTROLS



LASER LENS

Capsulotomy and iridotomy procedures often require the use of a laser lens, these are sourced independently by the user



SWITCH OFF

At The end of the case, press ON/OFF button and unlock wheels before unplugging the power

TROUBLESHOOTING

→ service@meridian.ch

DISPLAY	1	2	3	4	5
VIEW					
STATUS	Normal operation	Fail operation	Normal operation	Normal operation	Fail operation
ACTION	Set required parameters for Ready mode.	Reset unit.	Press icon to confirm message.	Press icon to confirm message.	Reset unit and in case of remaining Error call service.

Switzerland

Meridian AG
Bierigulstrasse 7
3608 Thun

Finland

Meridian Medical Oy
Elannontie 5
01510 Vantaa

Slovenia

Meridian Medical d.o.o
Plomljenova 8
1210 Ljubljana

Distributor Information

OPHTHALMALOGIC / MERIDIAN
Rick Behrens
0431 002 302